

Swann-Morton®


# INTRODUCING **KLEEN** BLADE MANAGEMENT SYSTEM SAFE. KLEEN. EASY.

THE SHARPEST, MOST PRECISE, MOST DURABLE BLADES YOU DON'T HAVE TO TOUCH

## DIRECTIONS FOR USE


### ATTACHING THE BMS

- 1 Slide handle **B** into the rear of the sheath **A** until the fitment engages with the slot in the blade.  
\*\*You will hear an audible click and feel a catch when correctly attached\*\*

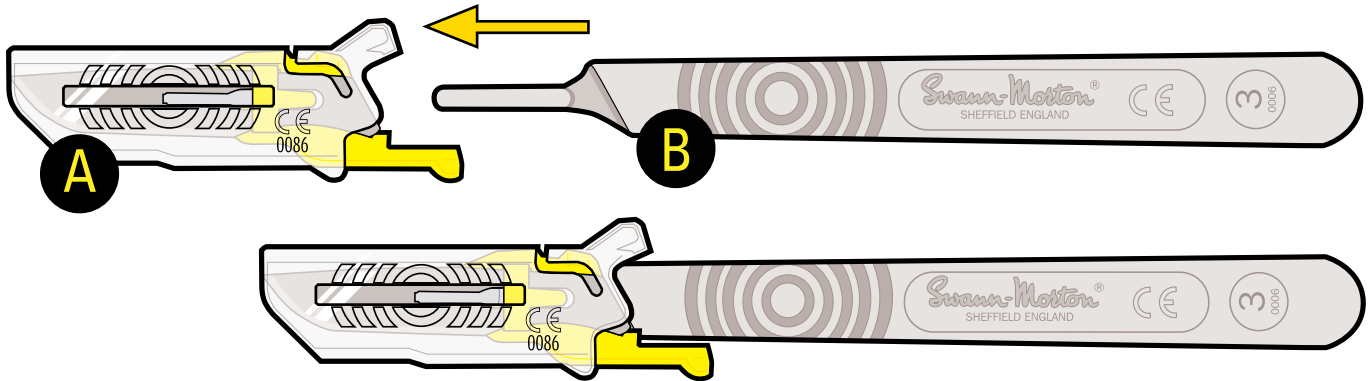


### EXPOSING THE BLADE

- 2 To reveal the blade **C** retract the sheath by pressing down on latch **D** and sliding the sheath backwards.  
\*\*You will hear another audible click and feel a catch when fully retracted\*\*

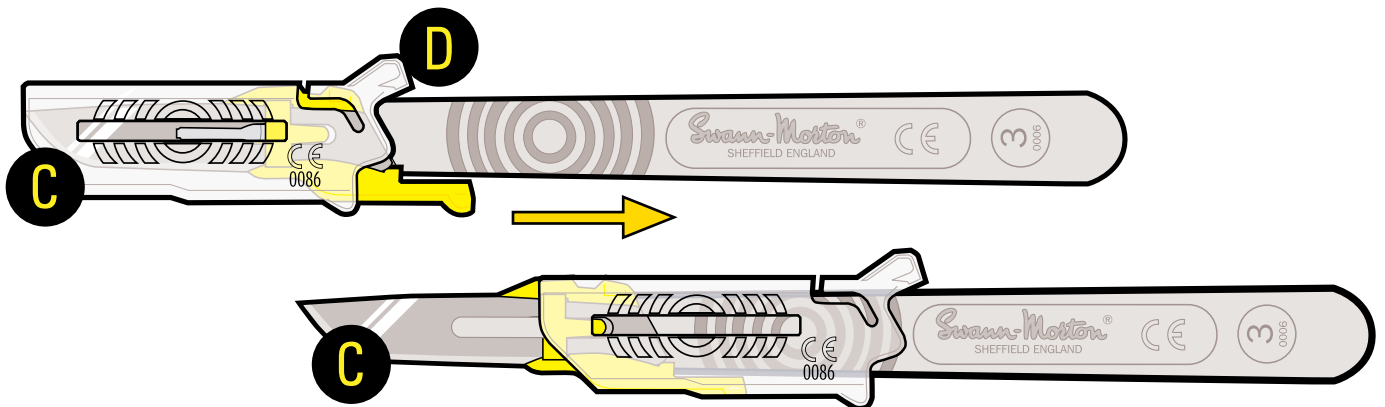


\*\*For procedures that require more longitudinal pressure the sheath can be locked firmly open by pressing and holding on tab **E** \*\*

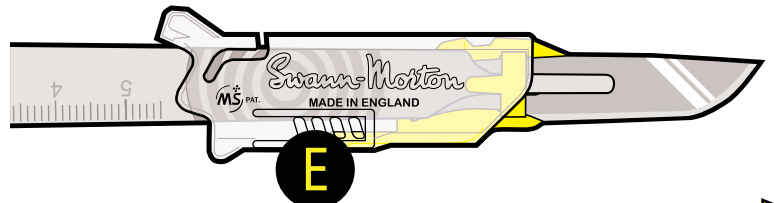

NEXT 


## CLOSING THE SHEATH

- 3 Release latch **D** while pushing sheath **A** forward.  
\*\*You will hear an audible click and feel a catch when correctly attached\*\*

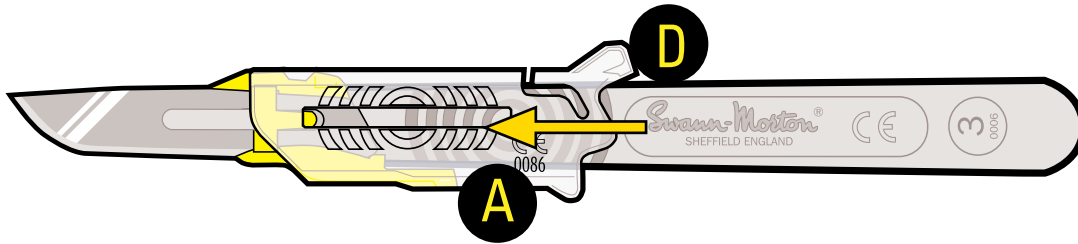

## DETACHING THE BMS

- 4 After closing the sheath, hold the scalpel in your right hand with the blade pointing downward, away from your body with the sharp edge facing to the right. With thumb on latch **D** and forefinger on blade-removing lever **F** push down and forward simultaneously.

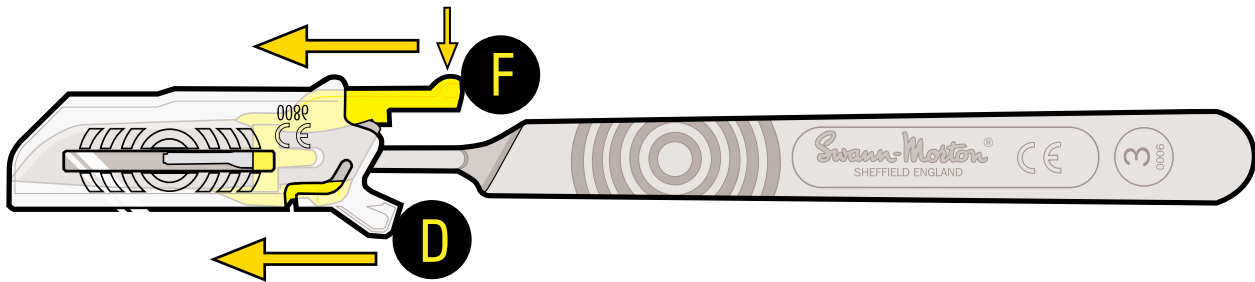

## DISPOSAL

- ! Be sure to dispose of all blades in an appropriate sharps container.



For a video demonstration of this product visit:  
<https://www.youtube.com/user/cincinnatiSurgical>

Call 800-544-3100 to set up a demo or request samples.  
Visit CincinnatiSurgical.com to learn more about all our safe surgical solutions.

Made in Sheffield | ISO 13485 Certified Production Facility



TRUSTED FOR OVER 70 YEARS